

If you **TRULY** care about your sample,  
let us help you **PROTECT** its integrity!

## C576 SecurTainer™ II TAMPER EVIDENT Specimen Containers



This model uses a different tamper evident concept when compared to the C575 Series. No locking tab is necessary when the tamper evident feature is used. Simply screw the cap on the container and the tamper evident sealing ring is automatically locked in place. When unscrewed, the ring is detached from the cap, showing clearly that the container was opened.

Also designed for collection, transport and storage of specimens, the polypropylene containers are safe to use even in adverse conditions. Containers are uniquely stackable, shatter resistant and are manufactured from virgin, polypropylene. These straight sided containers are available in many sizes from 20 to 120 ml. Non sterile. Containers and caps packaged separately in bags of 100.



Screw cap on until a clicking noise is heard. This is when the plastic ring is firmly seated and locked over the container threads. When removing the tamper evident screw cap for the first time, the perforation is severed, thereby ensuring easy recognition that the container has been opened.

For IVD use

Cat. #	Volume	Qty/Pk	Qty/Cs
C576-20MA	20 ml	100	500
C576-40MA	40 ml	100	500
C576-60MA	60 ml	100	500
C576-90MA	90 ml	100	400
C576-120MA	120 ml	100	300



For color coding purposes, use a Capinsert™ on top of the closure. Ten different colors are available. (see T345 Series). For more details and colors available please refer to page 18.



## C575-120MATI HydroTainer™ TAMPER EVIDENT Water Sample Container



Container made of polypropylene / Closure made of polyethylene

This 120 ml sterile container is ideal for collecting water samples from a multitude of sources prior to laboratory testing. It contains one 10 mg sodium thiosulfate tablet for reducing and dehalogenating the sample.

The uniqueness of the HydroTainer™ is that it incorporates a tamper evident screw cap ensuring peace of mind during transport or storage situations where someone might have manipulated the specimen. Supplied with I.D. label and tamper evident seal.

Cat. #	Volume	Qty/Pk	Qty/Cs
C575-120MATI	120 ml	100	300